

INHALO[®]

SECOND GENERATION VALVE

Medical Oxygen

Step-by-step



- 1 Check the product label to ensure you have the correct gas.

! Always use the label as the primary means of identification.

- 2 Check there is sufficient contents for your requirement.

- 3 Turn the flow selector to zero.

- 4 Remove the light grey tamper evident seal. This cover is recyclable. Proceed to open the light grey hinged cover.

- 5 Connect equipment and/or standard tubing.

! Check Oxygen tubing makes a secure fit prior to use.

- 6 Slowly rotate the hand wheel in an anti-clockwise direction fully until 'ON' indicator is shown.



- 7 Rotate gas flow selector to desired flow rate.



For more detailed instruction go to:

www.inhalo.com.au

or scan QR code



BOC: Living healthcare