

INHALO[®]

FIRST GENERATION VALVE

Medical Oxygen




Step-by-step



5 Connect equipment and/or standard tubing.

! Check Oxygen tubing makes a secure fit prior to use.

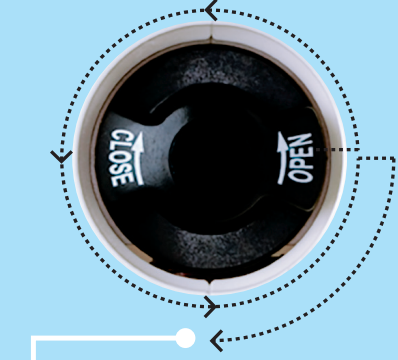

6 Slowly rotate OPEN/CLOSE hand-wheel in an anti-clockwise direction fully, then a quarter of a turn clockwise.

4 Remove the light grey tamper evident seal. This cover is recyclable. Proceed to open the light grey hinged cover.

7 Rotate gas flow selector to desired flow rate.



3 Turn the flow selector to zero.

2 Check there is sufficient contents for your requirement.

1 Check the product label to ensure you have the correct gas.

! Always use the label as the primary means of identification.

